



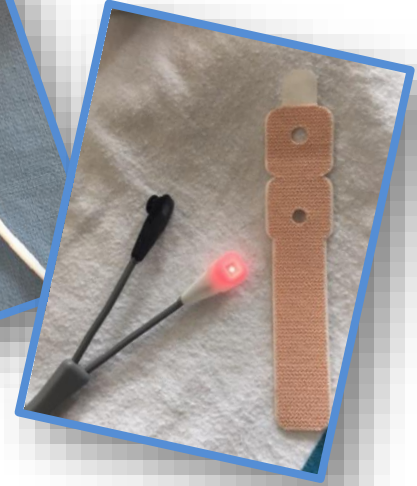

How to Apply the Reusable Pulse Oximetry Probe to a Neonate

1

Use a neonatal probe. Identify the light emitter and the light detector. You will also need a disposable wrap to secure the probe.

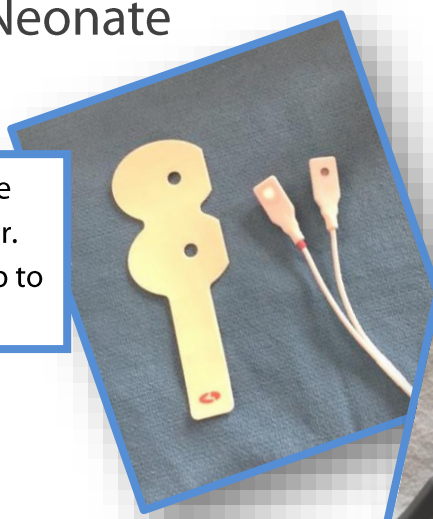

2



Select the site. For the hand, use the outer, non-thumb side. For the foot, use the thinner, non big toe side. The skin should be clean and intact.

3

Insert the probe 'bumps' into the wrap guide holes prior to application to hand or foot.

Apply the probe securely but not tightly, ensuring the light emitter and light detector are facing each other.

Clean the probe after use according to manufacturers' recommendations.

